


1 On/Off Switch Location


Press to turn on / Hold down for 5 seconds to turn off.

Please Note, All carts will power on automatically when plugged in to mains.


nano Located - Base of Power-slice

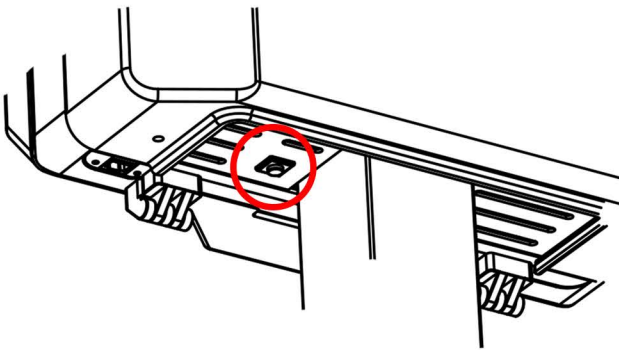

nova Located - Base of Power-slice

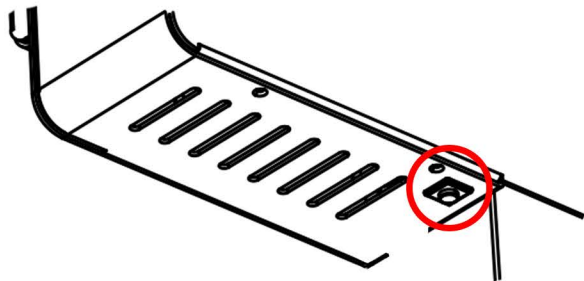


cube Located - Front Of Cart

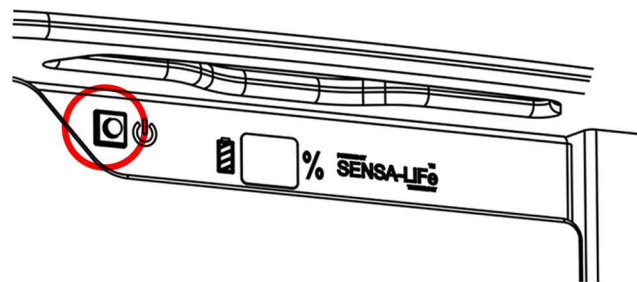

2 Battery Display Info. NANO & NOVA


When the cart's power level is low, the LED bar will flash red, and the cart will emit a beeping sound to indicate that it requires charging.

While the cart is charging, the 'power bar' will gradually fill up with a green indicator.

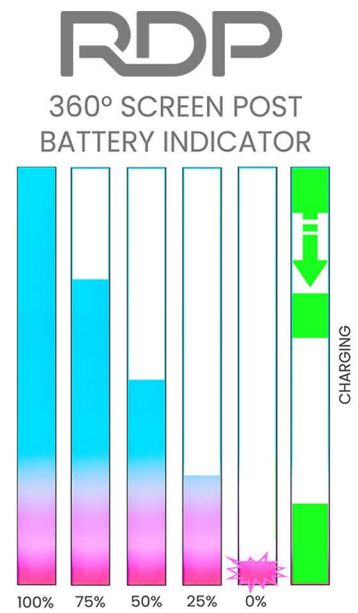


Battery Display Info. CUBE

When the cart's power is running low, an alarm will sound, and the indicator will flash red, indicating the cart needs charging.

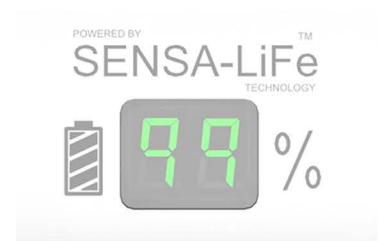


CART IS NOT CHARGING AND/OR PC TURNS OFF WHEN UNPLUGGED

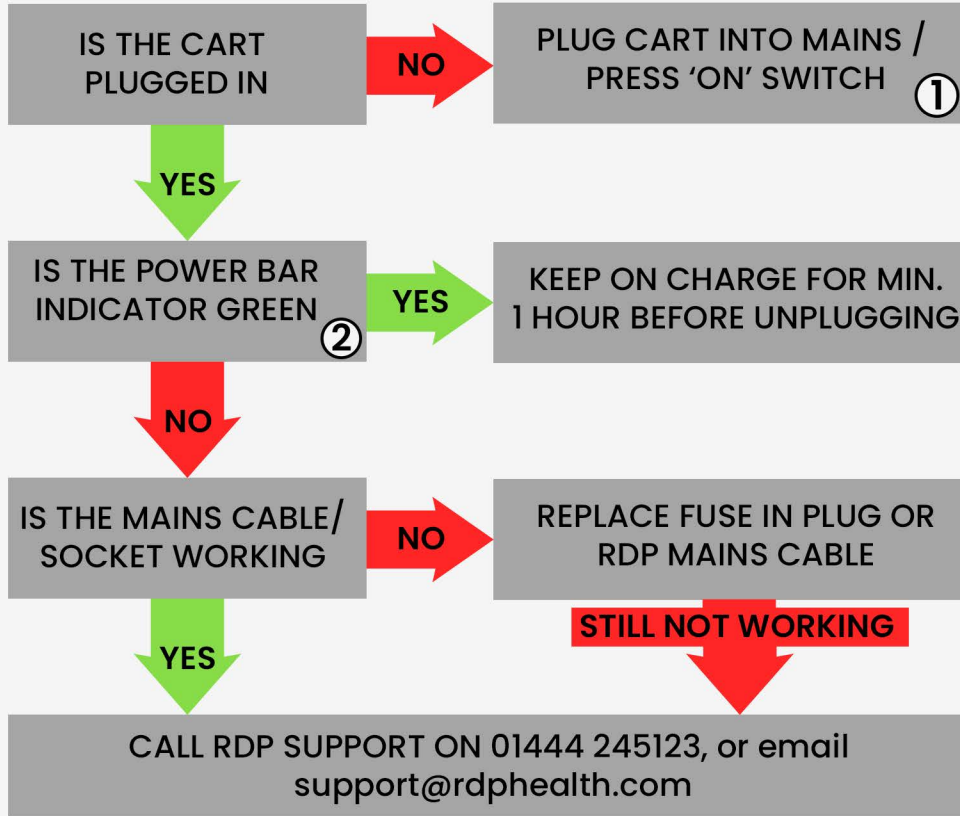


IMPORTANT.

KEEP RDP COILED CABLE CONNECTED TO CART AT ALL TIMES.

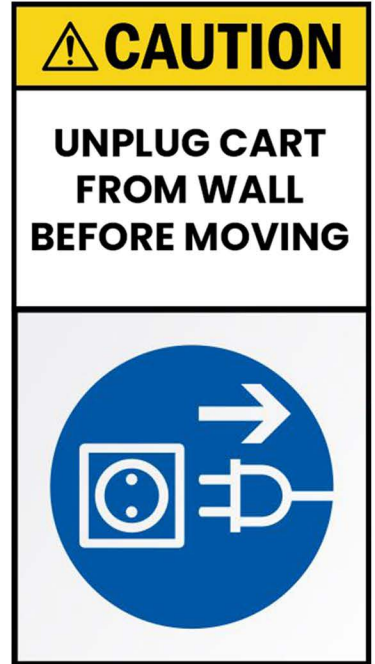


ELECTRIC HEIGHT ADJUSTMENT NOT WORKING

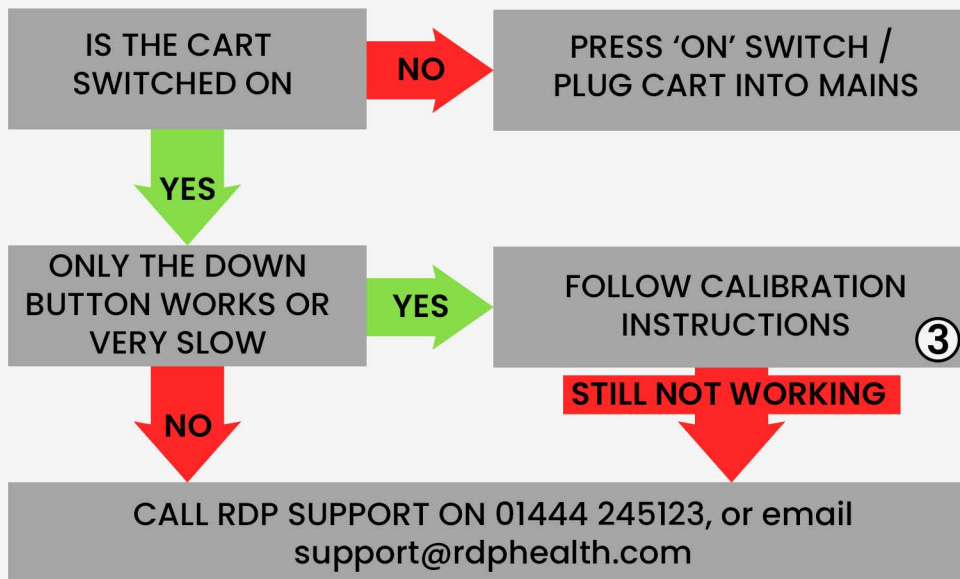

Strut Controller

Press and hold the corresponding height adjustment button in order to raise or lower the work surface.

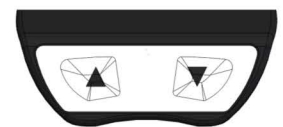

③ Strut Calibration

Lower the cart to its lowest position, then press and hold the 'down' button for approximately 5 seconds until you hear a distinct clicking sound. The strut should then operate at an increased speed.

